


# **INSTRUCTIONS FOR YOUR SARS-COV-2 RAPID ANTIGEN HOME TEST KIT**

# PLEASE READ ALL INSTRUCTIONS BEFORE COLLECTING YOUR SAMPLE

## **YOUR KIT CONTAINS**



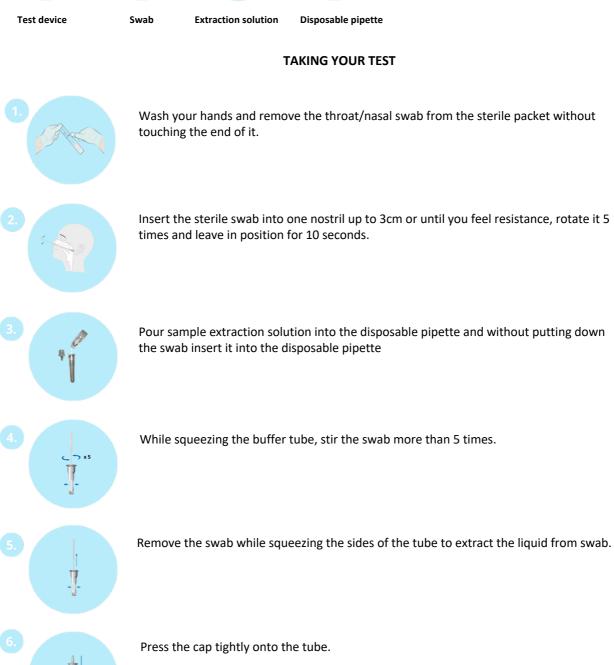



# INSTRUCTIONS FOR YOUR SARS-COV-2 RAPID ANTIGEN HOME TEST KIT





Apply 3 drops of extracted sample to the specimen well of the test device and read the results at 15 to 30 minutes. Not before 15 minutes.

### **RESULTS**



A coloured line appears in the top section of the result window to show that the test is working properly. This is the control line (C). Even if the control line is faint, the test should be considered to have been performed properly. If no control line is visible the test is invalid. In case of a positive result, a coloured line appears in the lower section of the result window. This is the line (T). Even is the test line is very faint or not uniform, the test result should be interpreted as a positive result.



To receive your results please take a photo of test device which must include:

- A clear result on the device
- The Booking Reference Number
- The LFD serial number of the cassette
- The BRN in the photo matches the BRN of the Order
- The LFD Serial Number in the photo matches with the Serial Number entered

### in the account

To upload the picture login into your account, click on UPLOAD ANTIGEN TEST PHOTO, upload the photo and enter the antigen LFD serial number. Results for FTF LFD are 30 mins – 2 hours during business hours.

The Rapid COVID-19 Antigen Self-Test is a lateral flow test for the qualitative detection of nucleocapsid protein antigen from SARS-CoV-2 (Coronavirus, or "COVID-19") from direct anterior nasal (nares) swab directly from individuals who are both symptomatic and asymptomatic. Nasal swab samples from individuals aged below 12 years should be collected by or under supervision of adults. It is suggested that individual above 70 years of age should be tested with assistance if needed. This test is intended to aid in the rapid diagnosis of Coronavirus infections. If symptoms persist despite negative test results it is mended to visit a healthcare professional to seek follow

### Safety Precautions

Only use the equipment provided to take your sample.

Kit to be used for intended use only.

Keep out of reach of children.

Warnings
For in vitro diagnostic use only.
The test device should remain in the sealed pouch until use.

Do not use the test kit past its expiration date. Swabs, tubes, and test devices are for single use only

Do not interchange or mix components from other kits.

Testing should only be performed using the swabs provided within the kit

To obtain accurate results, do not use visually bloody or overly viscous (thick, sticky) samples.

Specimens must be processed as indicated in the Test Procedure section of this Product Insert. Failure to follow the instructions for use can result in inaccurate results. Inadequate or inappropriate specimen collection and storage can affect results. Use in very humid areas or when temperature is above or below 15-30°C can adversely affectively result.

Collect kit components and swab samples in a plastic bag and dispose of as household waste. Do not move the test device after applying the solution

The kit can be stored at room temperature or refrigerated (2-30°C). Do not freeze any of the kit components. Keep away from direct sunlight.

Test devices that have been outside of the sealed pouch more than 1 hour should be discarded. Close the kit box and secure its contents when not in use.

Consult instruction for use  $\prod$ i Use by IVD For in vitro diagnostic use only Manufacturer **Authorized Representative** EC REP Do not reuse 2 Catalog# REF LOT Lot Number Keep out of sunlight 4 Store between 2-30°C -1 Keep dry Damaged package Caution Ţ) Sterile EO STERILE EO



H-Guard (China) Co., Ltd. 6th Floor, Building B1, No.14, Jinhui Rd., Pingshan District, Shenzhen, China Tel: +86 755 89305989 nerservice@h-guard.com.cn

EC REP

SUNGO Europe B.V. Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Swab information



lwood Medical Care Ltd 1-2 Floor, 3-919 Yongzheng Street Jinzhou District Dalian, China, 116100

EC REP

CMC Medical Devices & Drugs S.L C/Horacio Lengo No.18, CP29006, Malaga, Spain