

INSTRUCTIONS FOR STD SWAB SAMPLE COLLECTION

PLEASE READ ALL INSTRUCTIONS BEFORE COLLECTING YOUR SAMPLE

YOUR KIT CONTAINS



ODDB OST (MB) SEC MB TACK MB TACK MB TACK MIDD Same Dirg 1/2/2 C C Same Rest MB TACK MIDD Same Dirg 1/2/2 C C C C C

Login into your account on <u>www.medicinesonline.org.uk</u> and activate your sample by clicking on **SAMPLE ACTIVATION AND RESULT**, then select your test day, enter your **pre-paid tracking number** and the **tube number** (see under barcode) then click on **SAVE**.

TAKING YOUR SAMPLE



Wash your hands and remove the swab from the package without touching the end of the swab.

Female: Insert the swab 3-5cm into your vagina and turn gently for 20 seconds. Remove carefully without snapping the swab. DO NOT URINATE AT LEAST 1 HOUR BEFORE TAKING THE SAMPLE.



Male: Insert the swab 2-3cm into your urethra and turn gently for 20 seconds. Remove carefully without snapping the swab. DO NOT URINATE AT LEAST 1 HOUR BEFORE TAKING THE SAMPLE.





INSTRUCTIONS FOR STD SWAB SAMPLE COLLECTION



Without putting down the swab, unscrew the cap from the sample tube, carefully insert the end of the swab into the tube and break it inside the sample tube.



Once the part of the swab is broken inside the sample tube, please firmly screw the cap.



Place the sample tube into the plastic case, seal tightly and put it inside the return bag provided.

Make a note of the return tracking number under the barcode on the postal tracking label, as shown above. Keep this in a safe place for future reference. Post your sample in a Royal Mail Priority Post box on the same day it was taken.



Before you post your sample, double-check that you have:

- Activated your sample. 0
- Pushed the collection tube lid until it clicks. 0
- Kept a record of the tracking number. 0

Intended Use

Liquid VCM™ recovers viruses like Chlamydia, Mycoplasma, Gonorrhoea, Ureaplasma and HPV. Small vial with 1.0ml medium

Compatible with molecular and cell culture techniques. This product contains no hazardous constituents, or the concentration of all chemical constituents are below the regulatory threshold limits described by European directive 1272/2008/EC and the Occupational Safety Health Administration Hazard Communication Standard 29 CFR 1910/1200 and the Canadian Workplace Hazardous Materials Information System (WHMIS).

Safety Precautions

Only use the equipment provided to take your sample. Kit to be used for intended use only. Keep out of reach of children.

Warnings Do not open the collection tube until time of intended use

Contraindications Not to be used by persons under the age of 18 years old

Storage Do not expose to direct sunlight. Store product at 5°C to 25°C Store in cool, dry conditions



MWE Corsham UK SN13 9RT

EC REP

Advena Ltd, Swatar 4013 MALTA

Inbox of symbols

i	Consult instruction for use
R	Use by
IVD	For in vitro diagnostic use only
** *	Manufacturer
EC REP	Authorized Representative
8	Do not reuse
REF	Catalog#
LOT	Lot Number
	Keep out of sunlight
-1	Store at 5°C to 25°C
STERILER	Sterile R
Ŕ	Biohazard