

# **INSTRUCTIONS URINE STI HOME TEST KIT**

# PLEASE READ ALL INSTRUCTIONS BEFORE COLLECTING YOUR SAMPLE

## **YOUR KIT CONTAINS**



### **BEFORE TAKING YOUR SAMPLE**



Login to your account on <a href="www.medicinesonline.org.uk">www.medicinesonline.org.uk</a> and activate your sample by clicking on SAMPLE ACTIVATION AND RESULT, then select your test day, enter Royal Mail Tracking Number, and tube number (number under the barcode) then click on save.

## **TAKING YOUR TEST**



Wash your hands and open the sterile collection sample.

Urinate a small amount into the toilet. Place the sample container under stream and continue to urinate and collect specimen. About half the sample container is enough.



Screw the lid on the sample container tightly. Do not touch the inside of the sample container or lid.



Place the sample container into the return bag. And post the bag in a Royal Mail Priority Post box. To find the nearest Priority Post Box please download the Royal Mail app or go to www.royalmail.com/services-near-you/