

INSTRUCTIONS FOR CHLAMYDIA/GONORRHOEA RAPID TEST SAMPLE COLLECTION

PLEASE READ ALL INSTRUCTIONS BEFORE COLLECTING YOUR SAMPLE

YOUR KIT CONTAINS

1. SWAB 2. TEST DEVICE 3. SAMPLE DILUTION A 4. SAMPLE DILUTION B 5. EXTRACTION TUBE










HOW TO COLLECT A SAMPLE


- 1. Wash your hands and remove the sterile swab from the package without touching the end.
- 2. Insert the swab 3-5cm into your vagina (Image A), or 2-3cm Urethra (Image B) and turn gently for 20 seconds. Remove carefully, without snapping the swab. DO NOT URINATE AT LEAST 1 HOUR BEFORE TAKING YOUR SAMPLE.






FEMALE (A)

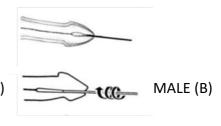

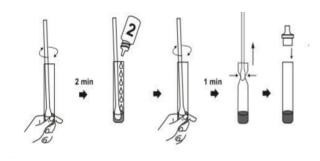

3. Drop 11 drops of sample dilution A into the extraction tube, put the swab into the extraction tube immediately, squeeze the tube and twist the swab for 15 times, keep the swab inside for 2 minutes before removal. Then drop 6 drops of sample dilution B, squeeze the tube and twist the swab for 15 times, keep the swab inside for 1 minute before removal. Discard the swab and cover the lid of the tube.

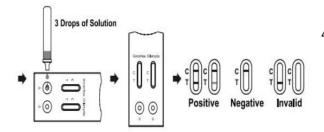

 Remove the test cassette from the foil pouch and place it on a clean, dry level surface. Add 3 drops of the extracted sample from the tube to the round Sample Well on the test cassette. Read results in 15-20 minutes. DO NOT INTERPRET RESULTS AFTER 25 MINUTES.



INSTRUCTIONS FOR CHLAMYDIA/GONORRHOEA RAPID TEST SAMPLE COLLECTION

RESULTS

Positive result

Any faint-to-dark red Test Line (T) along with a blue Control Line (C) indicates a positive result. A positive result indicates that the specimen is positive for the presence of Chlamydia antigen.

Negative Result

Only the blue Control Line (C) indicates a negative result. A negative result indicates that the specimen is a presumptive negative for the presence of Chlamydia antigen or it is below the detection level.

Invalid Result

The test result is considered invalid if the blue Control Line (C) is not visible within 10 minutes.

RECEIVING RESULTS

To receive your results please take a photo of test device and upload it in your account.

To upload the picture login into your account, click on UPLOAD STD TEST PHOTO, upload the photo.

Results for Chlamydia/Gonorrhoea Rapid Test are 30 mins – 2 hours during business hours.